

TECHNICAL DATA 46

<i>ATTITUDE</i>	<i>Manual</i>	<i>Hybrid</i>	<i>Power</i>
Max. user weight	100 kg	100 kg	100 kg
Front frame width (adj. + 2 cm)	26 – 46 cm	26 – 46 cm	26 – 46 cm
Weight of parts at chair	170 g	170 g	170 g
Total bike weight	14,5 kg	21,5 kg	15,3 kg
Wheel size	20"	20"	16"
Max motor speed	–	23 kph	6 kph & 23 kph
Motor	–	250 W	250 W
Battery	–	14,5 Ah	9 Ah & 13,4 Ah
Gearing shifters	Trigger / Grip shift	Trigger / Grip shift	–
Gearing speeds	8 & 30	10	–
V-crank width	470 mm	470 mm	–
Crank length	170 mm	170 mm	–
Handle bar width	–	–	570 mm
Brake	Disc (160 mm)	Disc (160 mm)	Disc (160 mm)
Parking brake	yes	yes	yes
Frame colour	31	31	31



For further information on product options please check the order form on our website.
You can view this document also in a larger font on a PDF-file at www.SunriseMedical.eu



Sunrise Medical GmbH & Co. KG

Kahlbachring 2-4
D-69254 Malsch/HD
Tel.: +49 (0) 72 53 / 9 80-0
Fax: +49 (0) 72 53 / 9 80-222
E-Mail: International@sunrisemedical.de
www.SunriseMedical.eu



ATTITUDE PURE FREEDOM



A COMPLETE ATTITUDE FAMILY

Experience freedom like never before with the new Attitude add-on wheelchair bike

Whether you want to exercise or to enhance your mobility there is an Attitude add-on bike perfect for you. The Manual version will offer you a lightweight and manoeuvrable package. If you would need some power support when you get tired the Hybrid will be your Attitude. With the Attitude Power you will have full power mobility.



ATTITUDE Manual

THE SPORTIVE





ATTITUDE Hybrid

THE ALL-ROUNDER



ATTITUDE Power

THE STRONG



READY TO GO IN FEW SECONDS


The Attitude bike's clever docking mechanism attaches to your wheelchair in a matter of seconds... and it's just as quick to remove. Ideal for when you don't have time to lose or are in a hurry.

The docking mechanism on the Attitude bike is so simple, it just locks securely to the front of your wheelchair – without any need for parts or cluttered wiring around your knees. So there's nothing coming between you and the comfort that you've come to expect from your wheelchair.



After de-docking only two small pins (170 g) remain and your wheelchair.

- 1 Lock in the docking pins.
- 2 Swing the upper docking connector downwards.
- 3 Lock the connection by pulling the red lever.

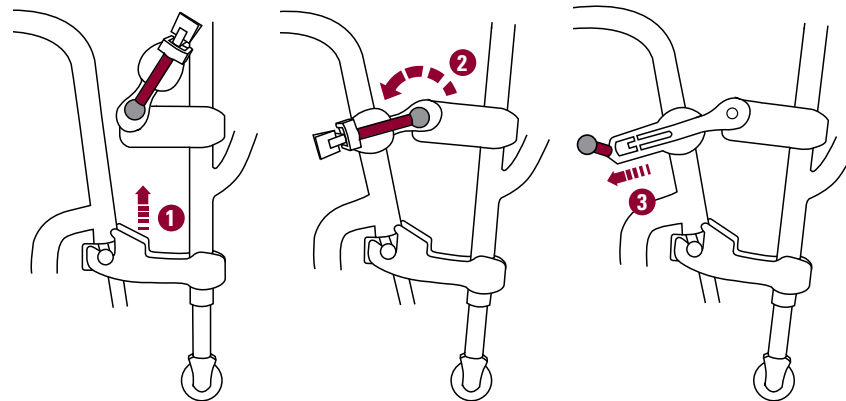

→ Ready to run the Attitude docking video! Check it out!



MORE FREEDOM

The freedom you'll enjoy with the Attitude bike not only applies to when you're biking. The ergonomic handle and support wheels will help you to manoeuvre the Attitude bike easily with just one hand. So whether you're de-docking your bike or storing it away, it's effortless no matter the situation.



PURE STYLE AT EVERY TURN

The lightweight and stylish Attitude bike will perfectly compliment your Quickie wheelchair. With its streamlined modern design and a selection of 31 eye-catching colours, you can easily individualise your Attitude bike to fit your own unique personality.



ATTITUDE

MANUAL – THE SPORTIVE



The Attitude Manual is ideal for those looking to improve their strength and overall stamina through recreational cycling.

20" front wheels and the 30 speed gearing provide an exceptional driving performance which is perfect for all terrains. And the powerful disk brake allows a smooth speed control. Lightweight and easily manoeuvrable, it's the perfect compliment to your active wheelchair.

Manual



// DUAL DRIVE (3X10)

30 speed gearing is perfect for all terrains. The disc brake allows a smooth speed control.



// GRIP SHIFT

The easy one hand operation grip shift allows quick and easy gear shifting.



// BOTTLE HOLDER

This bottle holder can hold all standard bottles (available for all Attitude hand bikes)



// PARKING BRAKE

Easy operating parking brake with lock in function as standard on all Attitudes.

ATTITUDE

HYBRID – THE ALL-ROUNDER



The Attitude Hybrid combines all of the health benefits of manual cycling with the reliability of powered propulsion. Helping to reduce fatigue, the powered motor can be engaged whenever you encounter a tricky hill-climb or just to provide you with that little extra speed when you need it.

Hybrid



// **LITHIUM-ION BATTERY**

14,5 Ah Lithium-Ion battery for improved range performance. The display shows the battery capacity remaining.



// **STRONG 250 WATT MOTOR**

The powerful 250 watt motor supports up to a 23 kph top speed, ideal for hilly terrain or longer distances.



// **USEFUL LOW RIDER CARRIER**

The low rider carrier can be used for any available standard saddle bags. This option is available for all Attitudes.




// **ERGONOMIC V-CRANK**

The ergonomic handle position prevents contact with your body particularly when manoeuvring around corners.

ATTITUDE

POWER – THE STRONG

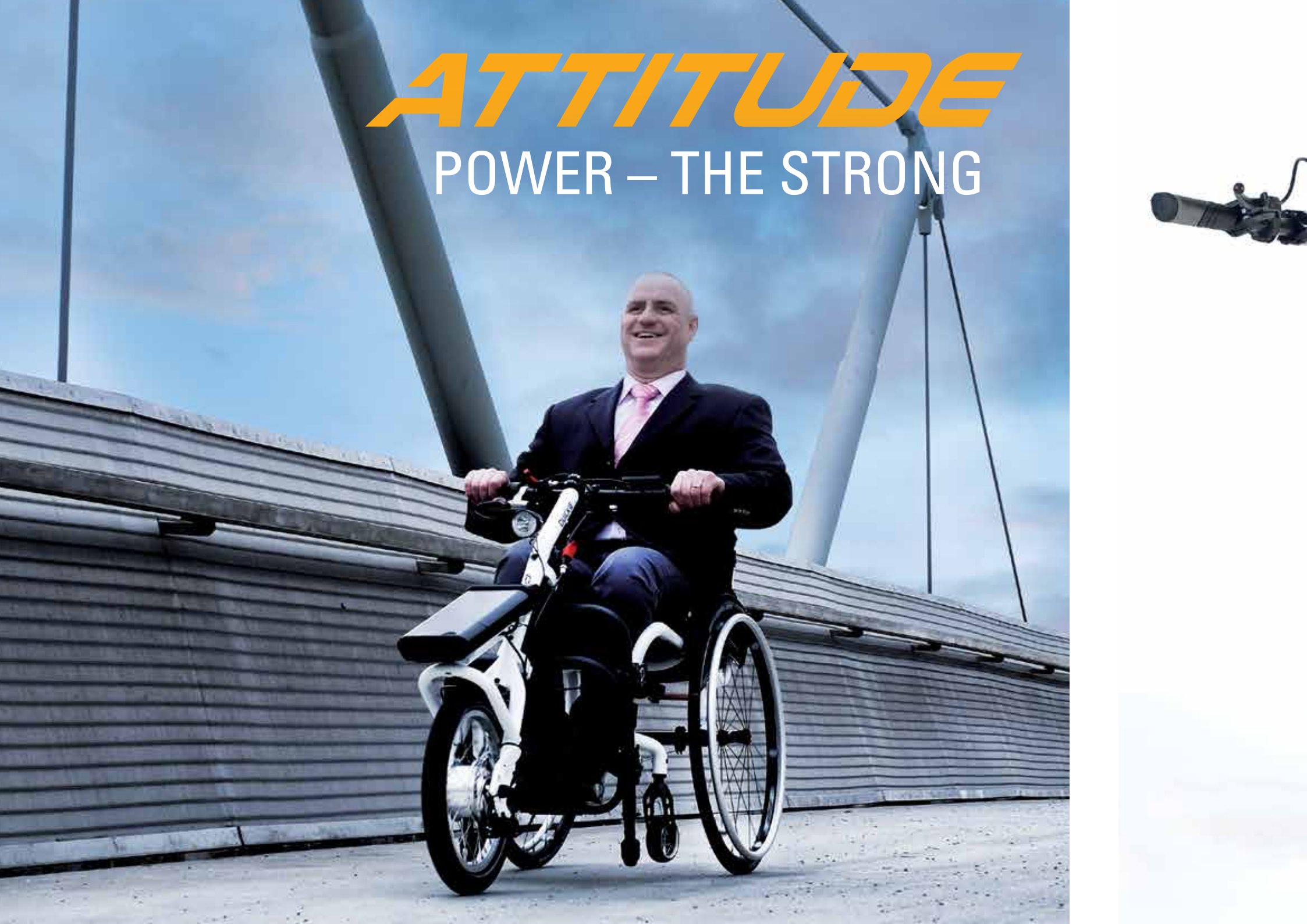

The Attitude Power provides you with 100 % powered propulsion from your comfort of your wheelchair. Relieving the strain on your upper-body, there's no need to worry about hill inclines or travelling distance – so you're free to relax and enjoy the ride.

Power



// **STRONG 250 WATT MOTOR**

Powerful 250 watt motor with 23 kph top speed.



// **ADJUSTABLE HANDLE-BAR**

The ergonomic handle bar is height adjustable via the stem. The display shows a range of driving options.



// **LITHIUM-ION BATTERY**

9 Ah or 13,4 Ah Lithium-Ion battery for large driving range. Lithium-Ion batteries have no memory effect.



// **INTEGRATED FRONT LIGHT**

The bright front light is powered directly from the battery.